

Model 610 Washer/High Level Disinfector Preventive Maintenance Schedule

Weekly (minimum, see the Model 610 Operation/Service Manual for more details):

- **Exterior surfaces:** Exterior surfaces should be wiped on a weekly basis with an appropriate germicidal cleaner. We recommend the use of a non-abrasive or low-abrasive cleaning agent whenever possible to maintain the appearance of the Model 610. The Control Panel should also be wiped on a weekly basis using a non-abrasive cleaner with a small amount of water.
- **Tank:** Clean the stainless steel tank weekly by performing a System Clean cycle. An alert will appear on the LCD screen as a reminder message when it is time to clean the system. Some manual cleaning is required during the cycle. An External Spray Nozzle is available to help clean the tank. The use of a sponge or Scotch-Brite pad and a cleanser such as Barkeepers Friend are effective cleaning agents. See **Effective Methods for Cleaning Stainless Steel** in the Operation/Service Manual. The frequency of cleaning may vary according to local water conditions and mineral content, frequency of use, or facility policy and procedure.
- **Interior surfaces:** Disinfect with anti-microbial disinfectant (CaviCide) and wipe down all stainless steel surfaces, including the lid and ports. Use a large cotton tipped applicator saturated with disinfectant for the water inlet port and the air dome port. Allow CaviCide to dry for 10 minutes before rinsing and drying.
- **Drain screens:** Weekly cleaning of the drain screen in the tank will prevent build-up of minerals and other residues. Failure to keep the drain screen clear of lint and other residues may result in standing water inside the tank and possible overflow. The drain screen can be cleaned with a toothbrush.

Note: To be effective as a disinfectant, all anti-microbial agents need to remain in contact with the surface for a length of time. Refer to the package label for instructions on proper use if not using CaviCide.

Biannually (minimum, see the Model 610 Operation/Service Manual for more details):

- **Preventive Maintenance:** It is recommended that Preventive Maintenance (PM) be performed every six months to keep the Model 610 in working order. This prevents the Model 610 from developing mechanical problems and wear that could lead to leaking, agitation inconsistencies, or other functional problems. A spare PM Kit should be kept on hand to avoid unnecessary down time. The Model 610 will display an alert message on the LCD screen when it is time to complete a PM.
- **Cleaning Solution Reservoir:** Every six months, the Reservoir should be rinsed out with water.

Listed below are available maintenance parts:

Part #	Description
020590-00	<i>CaviCide</i> disinfectant/cleaner – 24 ounces
020590-01	<i>CaviCide</i> disinfectant/cleaner – case of 12
020591-00	Cotton tipped applicator – box of 100
082600-00	PM Kit, Model 610
082380-00	PM Kit Spanner Wrench

[High Level Disinfection Systems for Healthcare](#)

Cenorin, LLC • 6324 South 199th Place • Suite 107 • Kent, WA 98032
Phone: 253-395-2400 • Fax: 253-395-2650 • Toll Free: 1-800-426-1042
FDA Registered • CSA Certified • ISO 13485:2003